

IRIS Installer

Advanced User Quick Start

1. Load and run the included Installer Disc prior to connecting the Camera.
2. Run either the 32 or 64-bit installer based on your operating system
3. Follow the instructions in the installer and choose the imaging software that the camera will be used with
4. Upon completion of the installation use the readme link for specific imaging software setup instructions
5. Connect the camera to a USB 2.0 port using the provided powered hub
6. To verify proper installation check that the camera is listed in the Device Manager as Iris under imaging devices.
7. If you experience any issues simply select Start - All Programs - Digital Doc - Utilities - Refresh Drivers

CAMERA
INSTALLER
POWERED BY
ELLY



Digital DOC
1.800.518.1102